Description

Variolink® N Try-In pastes are water-soluble glycerine pastes. They permit shade simulation of indirect composite and ceramic restorations (veneers, inlays, onlays, crowns) prior to final cementation using Variolink N luting composite. The ideal luting composite shade is determined with the help of the Try-In pastes. The translucency and shades of these pastes correspond to those of the thoroughly polymerized Variolink N luting composite.

Like Variolink N, the Try-In pastes are available in 7 shades:

Shades**Variolink N Try-In (DC)****A–D**

bleach XL	BL 1
transparent	T
white	A1
yellow (universal)	A3

Variolink N Try-In (LC)**A–D**

clear	–
+1	–
+2	–

Composition

Try-In pastes are composed of glycerine, mineral fillers, and colorants.

Side Effects

None known to date.

Interactions

Carefully remove Try-In paste from the restoration and the tooth before final seating. Otherwise, the bond between the restoration and Variolink N may be impaired.

Application

Trial insertion of the restoration is recommended before using the Try-In pastes, to check occlusion, proximal contacts and marginal adaption. After having selected the desired Try-In shade, apply a small amount of paste to the restoration or into the cavity. Carefully seat the restoration without pressure.

Important

Variolink N luting composite permits light-curing or dual-curing seating. When using the dual-curing method, simulation of the exact shade can be achieved by mixing Try-In paste shade A3 (simulates the Catalyst paste) with the second Try-In paste (simulates the selected Base paste). Pastes are mixed in equal parts.

Note

- Try-In pastes are water soluble. They are easily removed by holding the restoration under running water, by using alcohol or by placing it in an ultrasonic bath. Clean and dry the restoration thoroughly.
- Finally, the restoration is cemented using Variolink N (see Variolink N Instructions for Use).
- Try-In paste shades and the corresponding Variolink N luting composite shades vary slightly in the uncured state. The shade of all luting composites changes slightly during and after polymerization. Try-In paste shades are coordinated with those of the thoroughly polymerized luting composite.

Storage

- Do not use Variolink N Try-In after expiry the date.
- Store the material at room temperature.
- Shelf life: see expiry date.

Keep out of the reach of children!**For use in dentistry only!**

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

The Material Data Safety Sheet is available online at

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Ivoclar Vivadent Marketing Ltd.
Ivoclar Vivadent AB
Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.